

Disposable Respirators Seal Checks Infographic



User seal checks

Disposable Respirators

NEGATIVE AND POSITIVE PRESSURE

Cover respirator with both hands

Breathe **in** gently

Respirator should slightly **collapse**


If air leaks **in** around the face:

Cover respirator with both hands


Breathe **out** gently

Respirator should slightly **expand**


If air leaks **out** around the face:



Adjust straps or
Reposition face piece



REPEAT TEST UNTIL SEAL IS ACHIEVED



Helping you *Make it Safe*
safetyalliancebc.ca